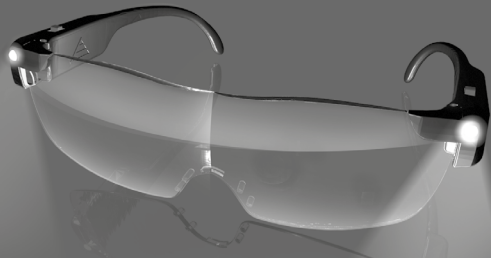
MAGICVISION PRO 300%

LED-LUPENBRILLE

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Deutsch

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English

Instruction Manual
from page 8

Français

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Item number: 2467

Dear customer, dear customer,

we are pleased that you have chosen this product. If you have any questions, contact customer service via our email address:

kundenservice@hsphanseshopping.de

Please read the instructions carefully before using the device for the first time and keep them safe. These instructions must also be handed over if the device is passed on. The manufacturer and importer accept no liability if the information in this manual is will not be noticed!

In the context of constant further development, we reserve the right to product, packaging or accompanying documents at any time.

All safety instructions are marked with this symbol. Read these carefully and follow the safety instructions to avoid personal injury and damage to property.

INTENDED USE

Thank you for your purchase of Magic Vision LED Magnifying Glasses, the magnifying glasses with which you can See things big, clear & bright! The Magic Vision LED magnifying glasses enlarge the objects without distortions, thus reducing the strain on the eyes and making daily tasks easier due to the bright illumination. With a 120° wide-angle magnification and good illumination, you can see even small details without the distortion. Exertion or strabismus ... and all that while your hands are free!

Special Features

UV 400 sun protection and blue light filter

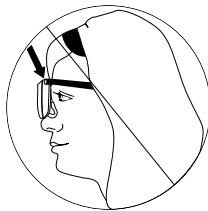
The Magic Vison Pro 300% LED magnifying glasses have a UV 400 filter protection. It blocks all UV rays in the range of less than 400 nanometers wavelength. Thereby all UVA, -B and -C rays dangerous to the eye. In addition, the LED loupe

brile is equipped with a blue light filter. This ensures that your eyes are protected from high-energy blue light. This light is suspected to cause eye diseases as well as sleep and concentration disorders.


USAGE

The Magic Vision LED magnifying glasses can be worn alone or over glasses or contact lenses. Magic Vision LED magnifying glasses should be worn on the tip of the nose, not in the same position as normal glasses.

WRONG USAGE

CORRECT USAGE

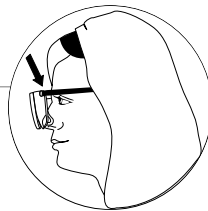
WEARING OVER GLASSES

When wearing with different glasses, including sunglasses, you should wear some. Leave a distance between the Magic Vision LED magnifying glasses and your glasses, to achieve the best focal length.

DETERMINE FOCAL LENGTH

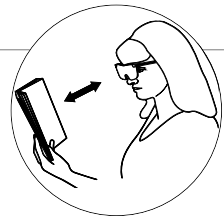Hold an object about 30 to 40 cm away from the Magic Vision LED magnifying glasses. Adjust the position until the object is in focus.

CLEANING AND CARE

To remove light dirt and fingerprints, use the cleaning cloth (supplied). For heavily soiled lenses, use water or a neutral detergent to clean them. DO NOT use a powerful detergent as it may scratch the lenses.

WARNINGS

Can only be used for close vision. Not for distance vision. Prescription-free spectacles available on the market do not replace prescription corrective spectacles or lenses or examination by an ophthalmologist. Regular eye exams by a qualified eye care professional are necessary to determine the health of your eye and your vision needs.

Please note: If you wear and use the LED magnifying glasses, no vehicles may be driven or machines operated. In addition, LED magnifying glasses are not safety glasses.

LED FUNCTION

These glasses are equipped with two LEDs. Through these LEDs an optimal illumination is guaranteed. The LEDs can be switched on and off individually via the switch. The supplied USB cable can be used to charge the permanently installed LI-ION batteries. The cable is designed in such a way that both batteries can be charged simultaneously. Connect the USB cable to a suitable adapter and then connect one or both temples of the LED Loupe Glasses with the cable. The charging time is approximately four hours. The charging time depends on the charge level of the battery and the voltage used. A charge status of the glasses is not displayed. The optimum lighting duration of the LEDs is approx. eight hours. If you notice that the LEDs become weaker, recharge the goggles.

DELIVERY SCOPE

1x glasses, 1x USB charging cable, 1x glasses cleaning cloth, 1x glasses band, 1x glasses bag (bag)

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HSP Hanse Shopping GmbH