DE - Anti-Kopfschmerz Kühlmaske

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EN - Anti-headache cooling mask

The cooling mask combines cold therapy and compression pressure for effective relief from headaches, migraines, swollen eyes or tension.

Cold application:

1. Place in a freezer bag in the fridge for at least 1-2 hours.
2. Remove the cooling mask from the fridge and wipe off any moisture. Wear as a headband or cooling eye mask.
3. Wearing time: 15-20 minutes.

If symptoms occur, remove the cooling mask immediately. If symptoms persist, consult your doctor.

Care instructions: Wipe the product with a damp cloth. Lay flat and allow to air dry. Do not immerse in water. Only place the cooling mask in the fridge when completely dry.

NOTE: For external use only. Do not use directly on open wounds, severe swelling or sensitive skin. For children, use only under adult supervision. Keep out of the reach of children. Place the cooling mask in the fridge always in a freezer bag. Only use the product if it is in perfect condition! Check the product for damage before each use. If the gel comes into contact with skin or eyes, wash the area immediately with soap and water. In case of pain or discomfort, stop treatment and consult a doctor immediately.

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Suzhou Sunmed Co., LTD
Shanghai International Holding Corp.GmbH (Europe)
Imported by / Vetrieben durch GHZ Matra AG