Art.-Nr.: 48748

Weinberger GmbH & Co. KG · Simonshöfchen 38 · 42327 Wuppertal · Germany · www.weinberger-group.de

Faltbarer GehstockArt.-Nr. 48748DE

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



Faltbarer Gehstock | Art.-Nr. 48748Seite 3DE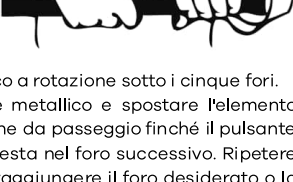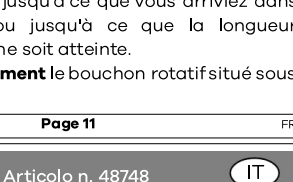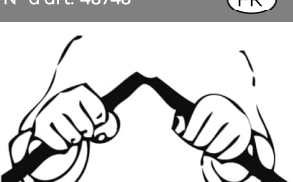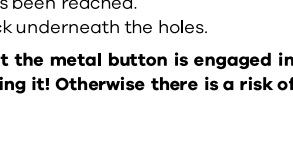

Foldable walking stickItem no. 48748EN

Thank you for purchasing our foldable walking stick. Please read the operating instructions carefully before using the product in order to become familiar with the features and functions in detail and keep them in a safe place !

Table of contents

Safety instructions.....	5
Cleaning and care.....	6
Use.....	6
Disposal.....	7
Technical data.....	8
Service.....	8

Safety instructions

- The walking stick may only be used for its intended use as a walking stick indoors or outdoors. Any other use is prohibited, as otherwise **damage** or **injury** may occur! No liability is accepted for damage resulting from improper use!
- Please fold the walking stick after use and store it safely. Protect the walking stick from direct sunlight or heat, strong temperature fluctuations, moisture as well as dust, dirt, falling and impact.
- The walking stick may only be used in normal ambient temperatures. **Not** in snow or on ice or in very hot environments.
- Do **not** attempt to repair the product yourself. The product does not contain any user-serviceable parts.
- Do not** make any modifications to the product and do not attach any additional elements.

Foldable walking stick | Art. no. 48748Page 5EN

Foldable walking stickItem no. 48748EN

- Always** check that the walking stick is stable and securely assembled before using it.
- Pay attention to the **maximum user weight of 136 kg!**
- This product is **not** intended for commercial use.

Cleaning and care

Remove dirt with a damp cloth and, if necessary, with a neutral cleaner and then wipe the product dry. Do **not** use any harsh cleaning agents, acids or alkalis, as these will severely attack and damage the material.

Use



Unfold:

Take the walking stick in both hands and pull it to its full length **until all four elements are locked into each other!**

Fold up:

Take the stick in both hands, pull it apart and fold the parts as shown in the picture. Insert the product in the folded state into the supplied oval plastic holder.

Foldable walking stick | Art. no. 48748Page 6EN

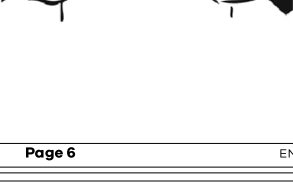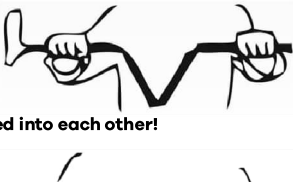Foldable walking stickItem no. 48748EN

Height adjustment:

The walking stick can be adjusted in length in five steps (from approx.84 to approx. 94 cm).





- Take the walking stick in both hands and fold it out first.
- Loosen the twist lock below the five holes a little.
- Press the metal button and move the top element of the walking stick until the metal button is engaged in the next hole. Repeat the process until you have reached the desired hole or until the desired length of the walking stick has been reached.
- Tighten the twist lock underneath the holes.

Important: Make sure that the metal button is engaged in one of the holes before using it! Otherwise there is a risk of injury!

Disposal

The walking stick does not contain any hazardous substances! Nevertheless, do not dispose of it with normal household waste, but dispose of it in an environmentally friendly way at your local disposal centre according to the materials.

Foldable walking stick | Art. no. 48748Page 7EN

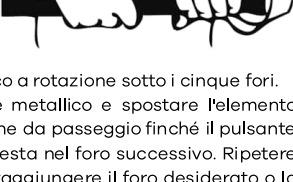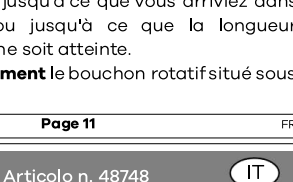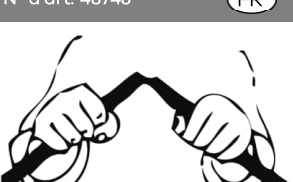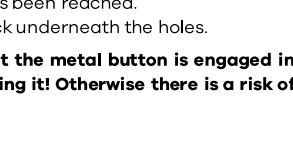Foldable walking stickItem no. 48748EN

Technical data

- Foldable into 4 parts
- Non-slip handle
- Height adjustable in five steps (from approx. 84 to 94 cm)
- Maximum user weight: 136 kg
- Product weight: approx. 420 g
- Dimensions (folded state): approx. 14.5 x 30 x 4.5 cm
- Dimensions (unfolded): approx. 14.5 x 84-94 x 4.5 cm

Service

Please feel free to send us an email to: service@weinberger-group.de

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